



















# OPERATING MANUAL

## WHEN RECEIVING YOUR CRYO-VACC™ UNIT

-  Check if the Cryo-Vacc™ Serial Nr is correct with the delivery email and temperatures as required on delivery document.
-  Check if the Cryo-Vacc™ screen is functioning and no warning notes appear on the screen.
-  Push the blue button on the screen to ensure temperature displayed on screen is correct.
-  Open up the cold chamber and ensure that the chamber is cold. Do not touch the aluminium sides of the chamber. Close the chamber lid.
-  Ensure your easy carry vaccine trays are in the cold chamber. Remove if required.
-  Check for general damage – notify the Cryo-Vacc™ team about delivery and request temperature reporting and address.
-  Once received and checked, please secure the Cryo-Vacc™ unit with use of wheel brakes.
-  Don't remove security tags on cryogenic chamber.

## DURING USE OF CRYO-VACC™

-  To access the cold chamber unlatch the locks on the case lid on the chamber side and open the lid.
-  Remove visible plug by pulling it out vertically using two hands. Place nearby while handling items in the cold chamber.
-  Food grade clear silicone paste is used in parts of the Cryo-Vacc™ and can sometimes be felt when opening the chamber.
-  When opening the cold chamber and handling of vaccines it is advised to always wear gloves.
-  Minimize opening time of the cold chamber. Leaving the chamber open for extended periods will result in rapid warming of the chamber and its contents.
-  The use of force when operating or handling the Cryo-Vacc™ may result in damage. If the lid or vaccines are stuck, move or rotate until items are free.
-  Vaccines to be placed in easy carry trays for easy handling.
-  Place the plug back with the painted section facing the thaw tray.
-  Close and latch the case lid before moving or leaving the unit again.
-  To check the temperature of the Cryo-Vacc™ press the blue button on the display. For further information and temperature logging contact your Cryo-Vacc™ rep or email [info@cryovacc.co.za](mailto:info@cryovacc.co.za)

